




UNIFLEX® TUR Collecting Bag

STERILE R


Intended purpose: OP device cover for collecting and draining liquids, for the duration of one application.
For mounting on the operating table.

1

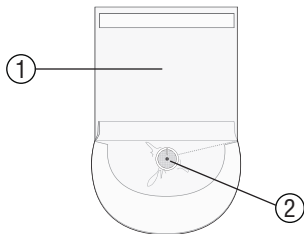


The TUR Collecting Bag (1) is unfolded. The discharge nozzle (2) points downwards (see also illustration 8).

2

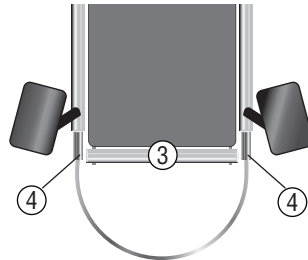


The holding rod (3) is removed at the spring brace holders (4).

3

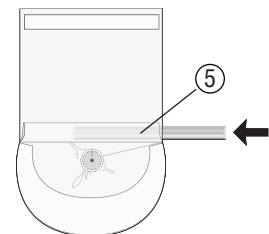


The holding rod can now be pushed into the seam of the bag (5).

4

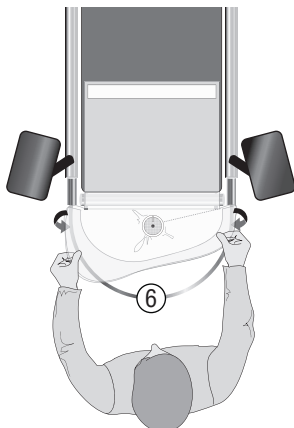


The rounded bag edge is fitted over the spring brace (6) and the holding rod is inserted into the spring brace holders.

5

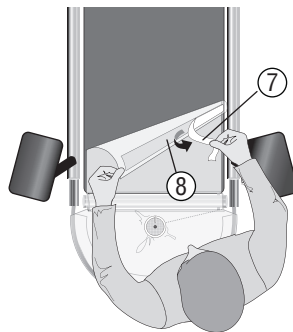

Remove the liner (7) from the adhesive tape (8).

6

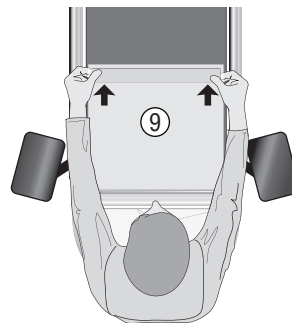


The area for resting on the table (9) is fixed to the operating table, making sure that there are no creases.

7




The TUR Collecting Bag is now affixed to the operating table. If fastened correctly, the bag cannot become detached from the table by itself.

8

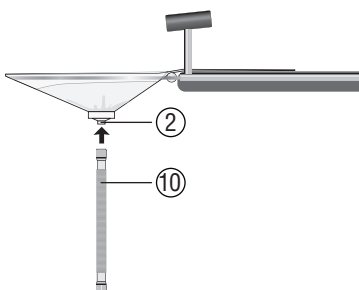

The corrugated tube (10) that is included in the scope of supply is pushed onto the discharge nozzle (2) of the Collecting Bag.


9

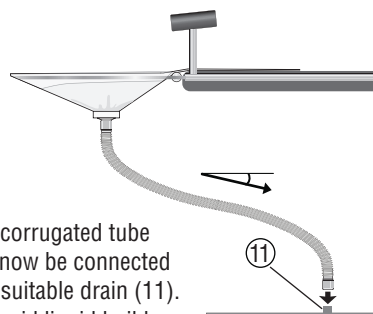

The corrugated tube can now be connected to a suitable drain (11). To avoid liquid build-up inside the drain hose, ensure a constant slope inside the hose.

Suitable adapters for the drain are available separately.

10

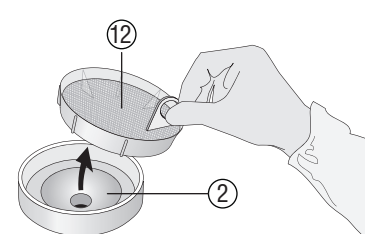


The filter sieve (10) holds back tissue fragments and thus prevents the conduit from becoming blocked. It can be removed from the discharge nozzle (2).






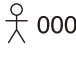













PACKAGE CONTENT		DETAILS
REF 37935	UNIFLEX® TUR Collecting Bag	Size: 50 x 45 cm
	Corrugated tube	Inside diameter of adapter: 22 mm, length extendable up to 200 cm
A detailed product specification is available on request.		

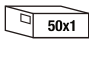WARNINGS and IMPORTANT INFORMATION

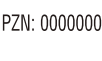-  Do not use if sterility is compromised or if the packaging is damaged
-  **CAUTION: Do not use damaged products!**
-  Do not reuse
-  Do not resterilize
-  **CAUTION: Not suitable for reuse**, as the required features may fail and cause an infection of the patient.
-  **CAUTION: The instructions for use apply only to the enclosed products.**
-  **CAUTION: Open peel pouch at top seam!**
-  Store in a dry place
-  Keep away from direct sunlight
-  **ATTENTION: To avoid liquid build-up inside the drain hose, ensure a constant slope inside the hose during installation.**
-  **CAUTION: To be used only by healthcare professionals!**
-  **CAUTION: Any serious incident that has occurred in relation to the device has to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.**

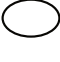OTHER SYMBOLS USED

-  Sterilized using irradiation
-  Use-by date
-  Latex-free
-  Consult instructions for use. The latest instructions for use can be downloaded at www.heisig.de
-  Number of consumption units per peel pouches
-  Personnel number of the manufacturing employee
-  Batch code JJMM0123
-  Manufacturer
-  Number of peel pouches in a carton
-  Date of manufacture
-  Article number
-  Pharmacy central number
-  The product is compliant with valid legal requirements for medical devices. Manufacturer is monitored by the notified body with number 0197.
-  Medical device
-  Single *sterile* barrier system
-  Single *sterile* barrier system with protective packaging outside
-  Environmentally compatible if properly disposed of. No toxic gases arise from burning. Dump without groundwater pollution.

